

IMPORTANT!

DISINFECTION INSTRUCTIONS

For ULTRASITE[®] Needle-free IV Systems

Three simple steps to flush or administer medication

1 WASH HANDS

- USE ANTIBACTERIAL SOAP.
- WEAR GLOVES FOR IV PROCEDURE.




2 DISINFECT VALVE TOP

- SWAB WITH ALCOHOL PAD: TWIST BACK AND FORTH IN A CIRCULAR MOTION FOR A MINIMUM OF 10 TIMES.
- ALLOW ALCOHOL TO DRY. (Drying time kills bacteria)



TWIST & CLEAN TOP & AREA INSIDE YELLOW CIRCLE

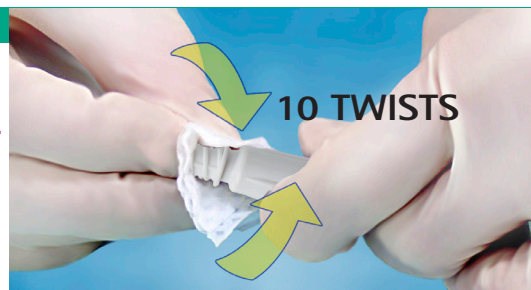

3 ATTACH & DISCONNECT

- IMPORTANT:**
- **DO NOT clamp line until syringe is disconnected.**
 - Remove any residual fluid upon disconnect with alcohol pad.



ALWAYS REMEMBER

- Swab valve top 10 twists with an alcohol pad before each infusion access.
- Flush before and after meds.
- Flush before and after blood draws.
- Disinfect valve top with 10 twists after each blood drawing process is complete.
- Upon completion of IV infusion follow institutional protocol for IV care and valve change.
- Always refer to package instructions for complete directions for use.

Rx Only. ©2006 B. Braun Medical Inc., Bethlehem PA. All rights reserved.

 **McLaren**

HOME INFUSION