

**ENDOSCOPE DISINFECTION / CIDEX OPA LOG SHEET**

PATIENT IDENTIFIER	EQUIPMENT or DEVICE ID	PROCEDURE	INSPECTION & LEAK TEST (+) PASS (-) FAIL	MEDIVATOR UNIT#	TODAYS DATE (mm/dd/yy)	DATE SOLUTION EXPIRES* (mm/dd/yy)	DATE TEST STRIPS EXPIRE** (mm/dd/yy)	TEST STRIP RESULT at 90 seconds (+) PASS (-) FAIL	TEMP OF SOLUTION IN ° F BEFORE USE (greater than 77° F (25° C) automatic; 68° F (20° C) manual)	CYCLE COMPLETED (Submerged 5 min. auto. OR 12 min. manual, 3 water rinses, one min. each, & alcohol flush)	PROCEDURE DONE BY (initials)

CIDEX OPA SOLUTION

\*NEW BOTTLE – MANUFACTUER’S EXPIRATION DATE  
OPEN BOTTLE – 75 DAYS FROM OPEN DATE  
CONTAINER – 14 DAYS FROM POUR DATE

TEST STRIPS

\*\* NEW CONTAINER - MANUFACTUER’S EXPIRATION DATE  
OPENED CONTAINER – 90 DAYS FROM OPEN DATE

**Cidex OPA solution needs to be tested and temperature checked PRIOR TO EACH USE**

**If any of the above measures are not met, high level disinfection process should be stopped, issue resolved, and process restarted.**