

TEE PROBE/RHINO-LARYNGOFIBERSCOPE DISINFECTION / CIDEX OPA LOG SHEET

PATIENT IDENTIFIER	EQUIPMENT or DEVICE ID	PROCEDURE	INSPECTION & LEAKAGE/ CURRENT TEST (+) PASS (-) FAIL	TODAYS DATE (mm/dd/yy)	DATE SOLUTION EXPIRES* (mm/dd/yy)	DATE TEST STRIPS EXPIRE** (mm/dd/yy)	TEST STRIP RESULT at 90 seconds (+) PASS (-) FAIL	TEMP OF SOLUTION IN ° F BEFORE USE (must be at least 68° F (20° C))	FULLY SUBMERGED 12 MINUTES	3 STERILE WATER RINSES, (1 MINUTE EACH)	PROCEDURE DONE BY (initials)

CIDEX OPA SOLUTION

*NEW BOTTLE – MANUFACTUER’S EXPIRATION DATE
OPEN BOTTLE – 75 DAYS FROM OPEN DATE
CONTAINER – 14 DAYS FROM POUR DATE

TEST STRIPS

** NEW CONTAINER - MANUFACTUER’S EXPIRATION DATE
OPENED CONTAINER – 90 DAYS FROM OPEN DATE

Cidex OPA solution needs to be tested and temperature checked PRIOR TO EACH USE
If any of the above measures are not met, high level disinfection process should be stopped, issue resolved, and process restarted.